


SDR[®] flow+ Bulk Fill Flowable

SDR[®] becomes SDR[®] flow+ and available in 3 new shades

Dentply Sirona's self-leveling bulk fill flowable SDR is changing name to **SDR flow+**.

In addition SDR flow+ is now available in three new shades: A1, A2 and A3.

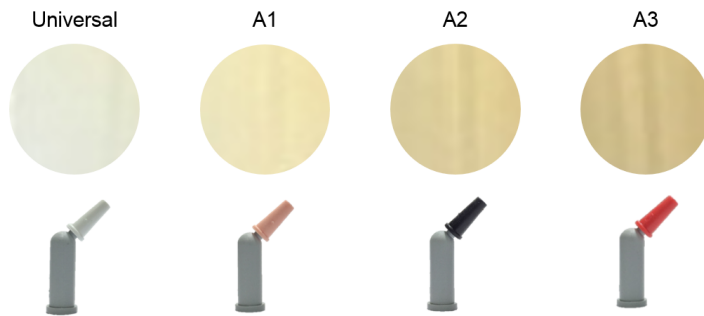

SDR flow+ is a self-leveling bulk fill flowable possible to place in a layer up to 4 mm in one go. SDR flow+ has a very low shrinkage stress and is unique to be supported by 5 year clinical data¹.

The range of indications for SDR[®] flow+ is getting broader and will also include class III and class V (in addition to class I and class II). The result is a self-leveling bulk fill flowable with wider range of indications and increased esthetics

Important: The light curing time for the three new shades are different vs the traditional universal shade. When applied in 4mm layer the curing time is 40 seconds for the new A1, A2 and A3 shades. (vs 20 sec for the universal shade)

¹ van Dijken JWV, Pallesen U, 2016: Posterior bulk-filled resin composite restorations: A 5-year randomized controlled clinical study; J Dent 2016 Aug;51:29-35.