



INSTRUCTIONS FOR USE

PRODUCT NAME	PRODUCT NUMBER
Dentorama Aluminium Instrument Tray System	TRA-010-1 ... TRA-010-5 (stand. trays) TRA-020-1 ... TRA-020-5 (perf. trays) TRA-030-1 ... TRA-030-5 (minitrays) TRA-015/025/035 (lids)
GENERAL INFO	
NOTE! Dentorama Trays are manufactured from coloured anodised aluminium. The colour can be damaged by contact with some chemical cleaners. Do not use chemicals too alkaline or too acid, a pH between 4 and 8 is recommended.	
RE-USE & END OF LIFE	
Repeated processing has no effect on these instruments. End of life is determined by wear and damage due to use.	
SPECIAL EQUIPMENT OR ACCESSORIES	
CLEANING INSTRUCTIONS	
<ul style="list-style-type: none">➤ DO NOT USE IN ULTRASONIC BATH➤ DO NOT USE IN DISHWASHER➤ BE CAREFUL USING DESINFECTING LIQUIDS –some are incompatible with aluminium	
The cleaning process aims to remove any waste from the trays and lids. Manual Cleaning: 1. Rinse excess soil from tray. 2. Leave the tray in Tray Cleaning solution as per instructions. 3. Rinse the tray. Automated Cleaning: 1. Use in autoclave or dry heater. Inspection: Visually inspect the tray. Discard damaged or broken trays.	
COMPATIBLE WITH	
DC1 (KOMET) Unisepta (UNIDENT) Dentasept (UNIDENT)	

**SVENSKA
DENTORAMA AB**

Manufacturer:
Svenska Dentorama AB
Gelbgjutarevägen 6
SE-171 48 SOLNA/SWEDEN

Phone: +46 8 734 99 75
Fax: +46 8 735 44 05
e-mail: svenska@dentorama.se
VAT-NO.: SE556253584801

Micro 10 (UNIDENT)
Gigasept (S&M)
Quitamet SA (SEPTODONT)
Detergyl SA (ZIZINE)

STERILISATION

Sterilize in Autoclave (134°), dry heat (180°) or chemiclave (132°).

STORAGE CONDITIONS

Store under dry conditions.

SHELF LIFE

No specific shelf life.

Latest update: WJ 2018-04-11



CERTIFIED
ISO 13485
Medical devices
Quality management systems

